

## RetractPLUS

### Instructions For Use

#### IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by dental professionals only. RetractPLUS is an astringent formulated to provide gingival retraction and haemostasis.

#### IDENTIFICATION

Product Name	RetractPLUS Retraction Paste 2 x 2 Syringes & 12 Tips
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

#### PRODUCT CODE - REF

010/1024 RetractPLUS Retraction Paste 2 x 2g Syringes & 12 Tips

#### INDICATIONS FOR USE

Taking an impression, Cementation and cavity preparation.

#### CONTRAINDICATIONS

Do not use on persons with known allergy to Aluminium Chloride. Do not use in patients with significant periodontal disease or furcation involvement.

#### PRECAUTIONS

- Do not insert cannula dispensing tip in to the sulcus.
- Be sure all paste is removed before taking impression to avoid inhibition of polymerisation of impression material.
- The dispensing tip is single use. Remove tip and dispose of properly after use.
- Recap the paste syringe before storin in the re-sealable foil bag provided – this may be used more than once.

#### INSTRUCTIONS FOR USE

1. Remove the cap from the syringe. Be sure to retain cap for subsequent storage of the material.
2. Place annealed luer dispensing tip onto end of the syringe tip, twist-lock into place. The soft tip allows for optimal access.
3. Rinse and air dry prepared tooth. Extrude paste slowly into the sulcus while maintaining the dispensing tip just above the sulcus and aligned approximately parallel to the axial plane of the tooth preparation. Be careful not to jam the dispensing tip into the sulcus.
4. Ensure you extrude sufficient material into the sulcus to achieve adequate tissue retraction.
5. Allow RetractPLUS paste to remain in the sulcus for at least one or two minutes. Often the marginal gingiva will grow pale when retraction is achieved.
6. After Haemostasis has been achieved the paste should be removed thoroughly rinsing using air-water syringe and saliva ejector or suction tip.
7. Examine treatment site to verify complete removal of the RetractPLUS prior to taking the impression.
8. Remove RetractPLUS dispensing tip and dispose of properly after use. Re-cap for storage in sealable foil bag.

#### WARNING!

Do not insert cannula dispensing tip into sulcus. Be sure all paste is removed before taking impression to avoid inhibition of polymerisation of impression material.

#### CAUTION

Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.



#### STORAGE

Keep away from direct sunlight, and store at room temperature (5-25°C).



#### Batch code/Expiry Date

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of the material. Please quote this batch number in all correspondence. The expiry date is shown in year, month format. Do not use the product after this date.



Perfection Plus Ltd  
6 Westwood Court  
Brunel Road  
Hampshire  
SO40 3WX, UK  
www.perfectionplus.com



Perfection Plus EU Ltd  
The Black Church  
St Mary's Place.  
Dublin, Ireland,  
D07 P4AX  
Regulatory@perfectionplus.com



Advena Ltd  
Tower Business  
Centre  
2nd Floor Tower  
Street  
Swatar, Malta

